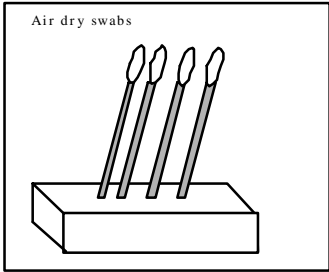
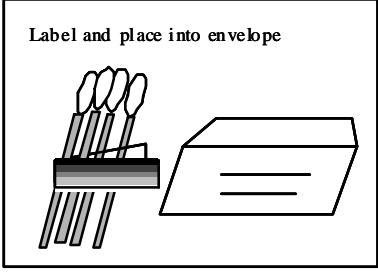


PATERNITY LEGAL HOME SPECIMEN COLLECTION INSTRUCTIONS

The kit you receive will contain the appropriate materials and paperwork for sample collection.

Drawsites: **Wear gloves** and follow appropriate biohazard collection procedures.

Buccal Cheek Swabs	<ul style="list-style-type: none"> • Collect, label and package swabs from one individual at a time. • Have the client rinse the inside of the mouth with drinking water. • Remove a sterile swab from the appropriate envelope (pink = Mother; yellow = Child; blue = Tested Man). • Rub the inside of one cheek vigorously; 25 times per swab while rotating swab. • Air dry swab in an upright position for at least 10 minutes (place in a clean empty glass or clean coffee cup with the cotton tip of the swab pointed up as in the diagram below). • Collect 4 swabs per person (2 per cheek). • Write the name of the person just collected on the swab label. Wrap the label around the 4 swabs in a flag-like manner, place dried swabs into the envelope provided and seal. Do not lick the envelope. • Initial the envelope and label with the client's name and date. Have the client initial the label. • Collect swabs from the next individual by repeating the above steps. • Ship samples the same day. Refrigerate if held overnight or freeze if held over the weekend. <div style="display: flex; justify-content: space-around; margin-top: 10px;"> <div style="text-align: center;"> <p style="font-size: small;">Air dry swabs</p>  </div> <div style="text-align: center;"> <p style="font-size: small;">Label and place into envelope</p>  </div> </div>
-----------------------------------	--